

Residential Hospice of Grey Bruce Inc. Nursing Manual	DIVISION: Nursing CATEGORY: Hospice Residence	POLICY NO: 1-R-18
ISSUED BY: Executive Director	SUBJECT: Goggle Use	PAGE NO: 1 of 1

POLICY: It is the policy of Residential Hospice of Grey Bruce that all employees and volunteers have a safe environment to work and volunteer in.

PROCEDURE:

1. Goggles must be worn when handling any chemical for cleaning, sanitizing or when pre-mixing disinfecting agents.
 2. Remove the goggles from their wrapper and grasping the outside of the goggles on the bridge and under the nose piece with one hand, and taking the strap in the other hand, fit the goggles over the eyes with the strap over the head and in back.
 3. When the task is finished, remove goggles by grasping the strap in back with one hand and with the other hand grasp the bridge and nose piece of the goggles, pulling the strap over and forward while pulling the goggles away from the face.
- Cleanse goggles in warm soapy water and then spray the exterior only with Virox allowing them to air dry. Return to plastic bag when dry

DATE APPROVED: April 2004

DATE REVIEWED:

DATE REVISED: April 2013

APPROVED BY:

April 2014

